


ultrasound probe you intend to use and connect it to the echocart.



Disinfect hands, wear gloves, and prepare the probe.

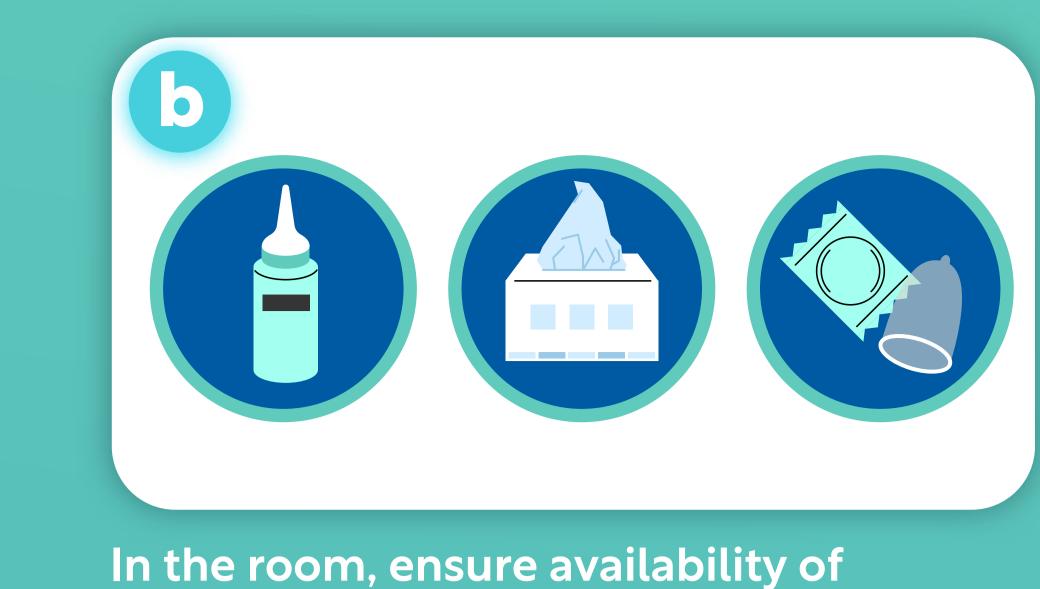


ultrasound gel, microfiber wipes and sheaths.




transvaginal probes.





Examination

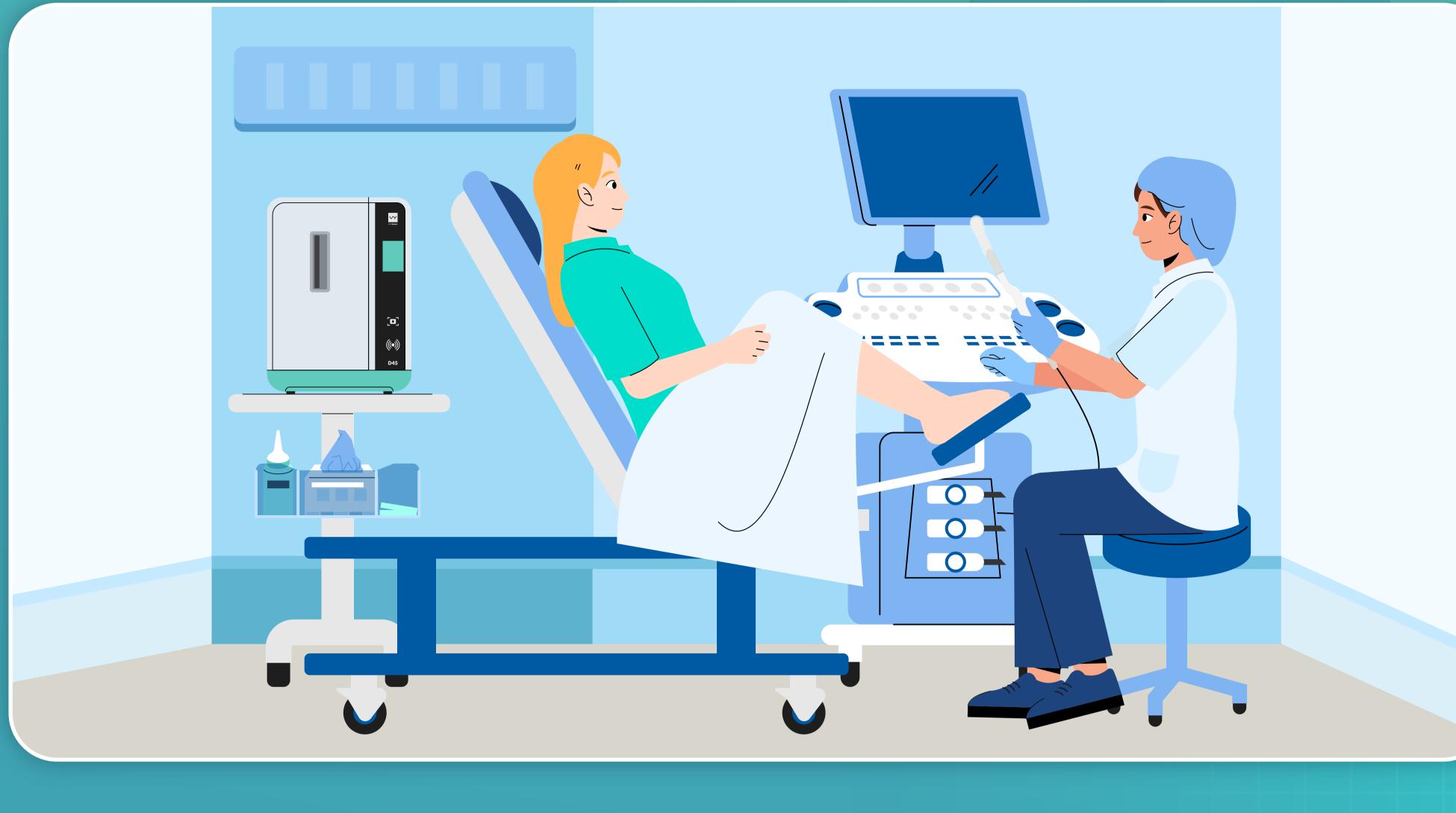




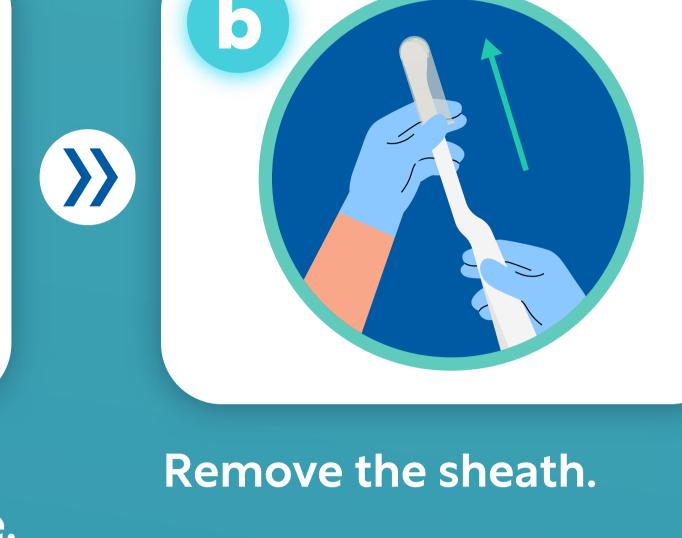


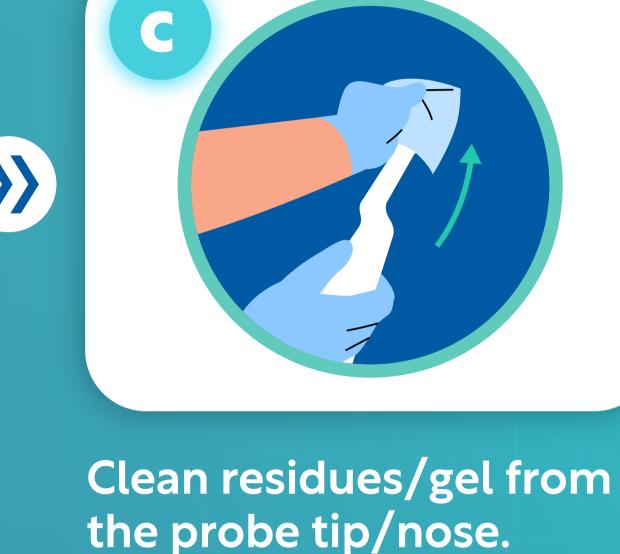


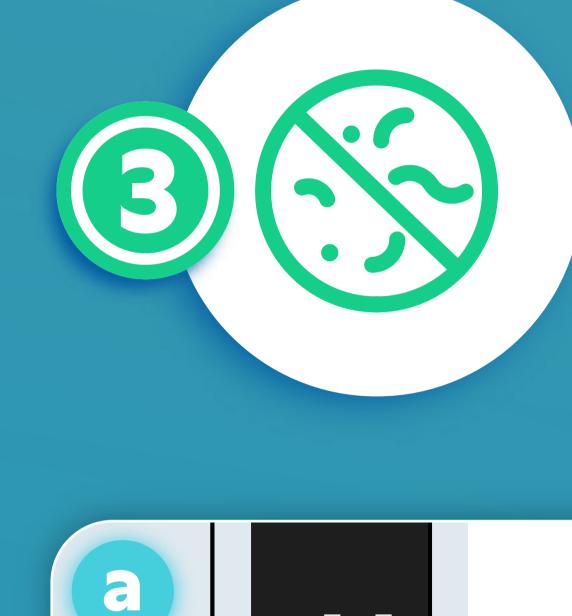

Disinfection Process

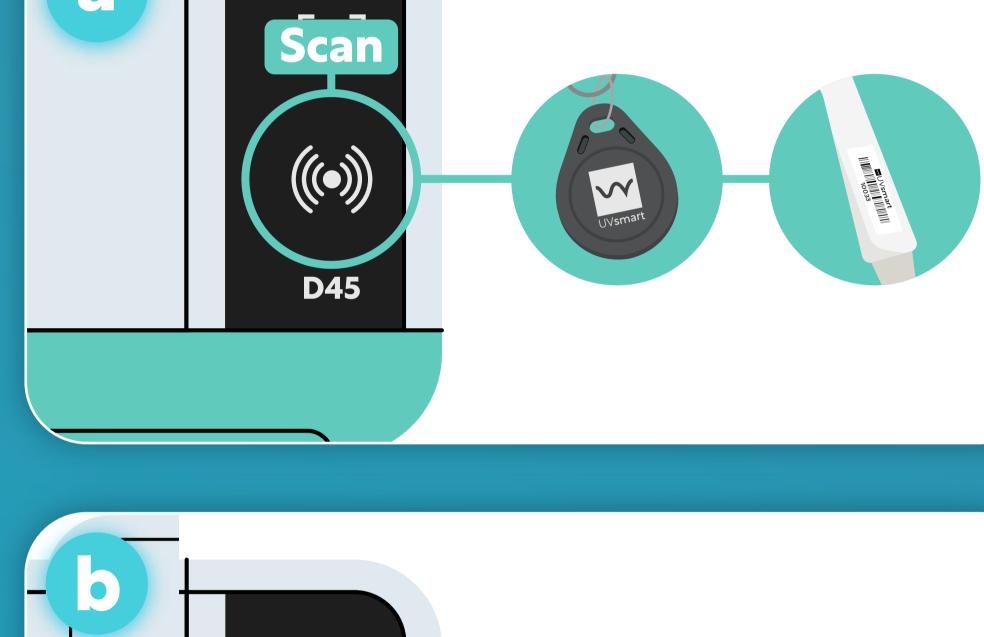


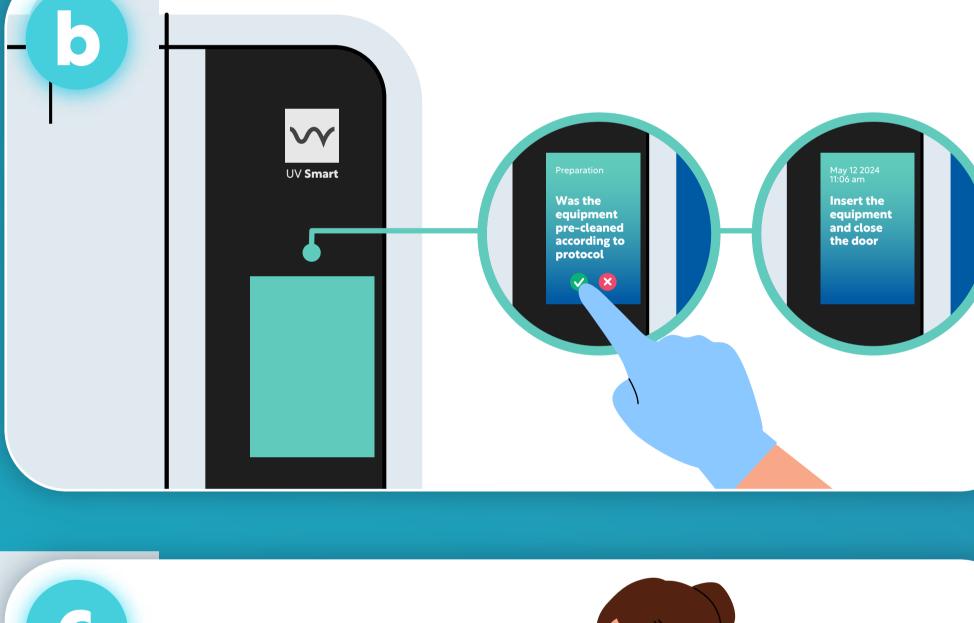


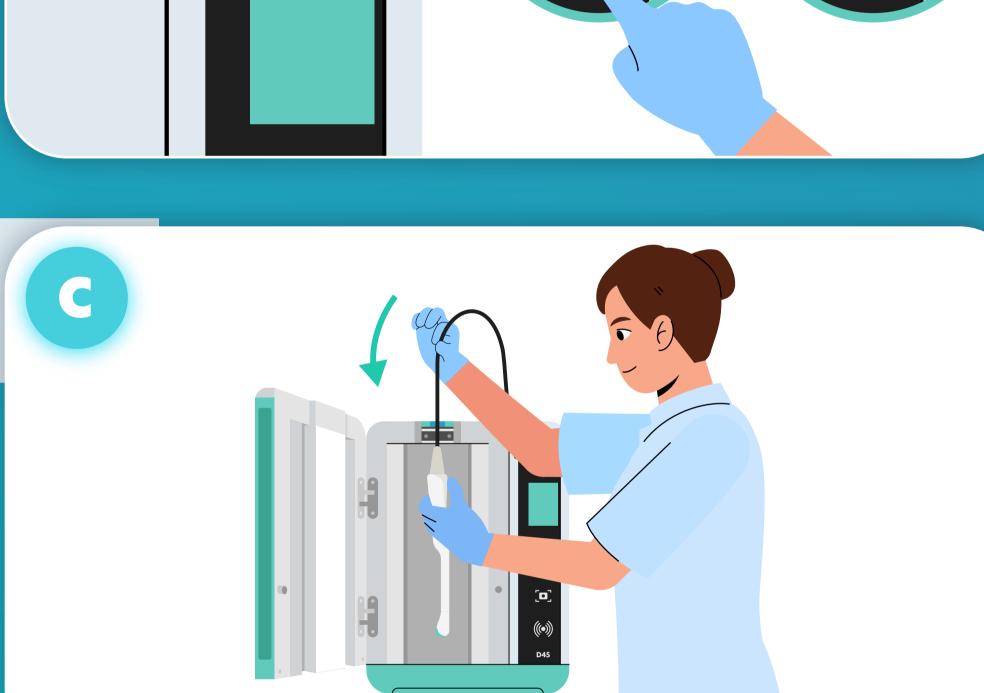



Few

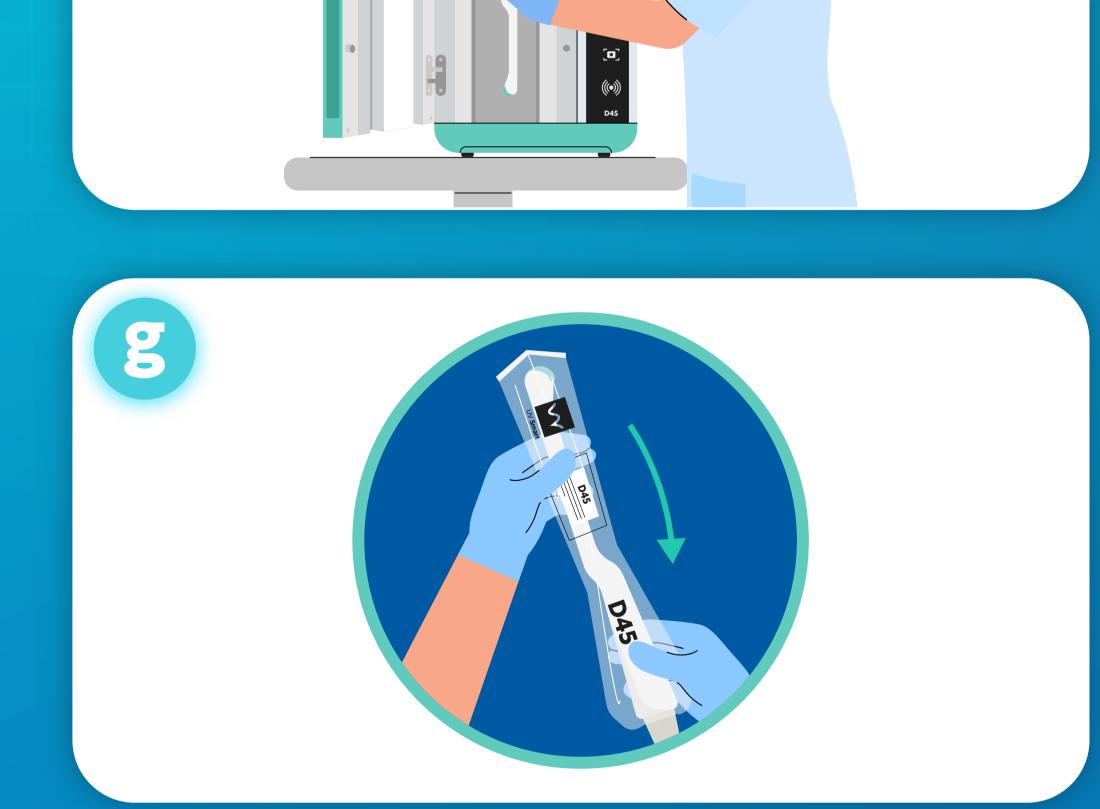

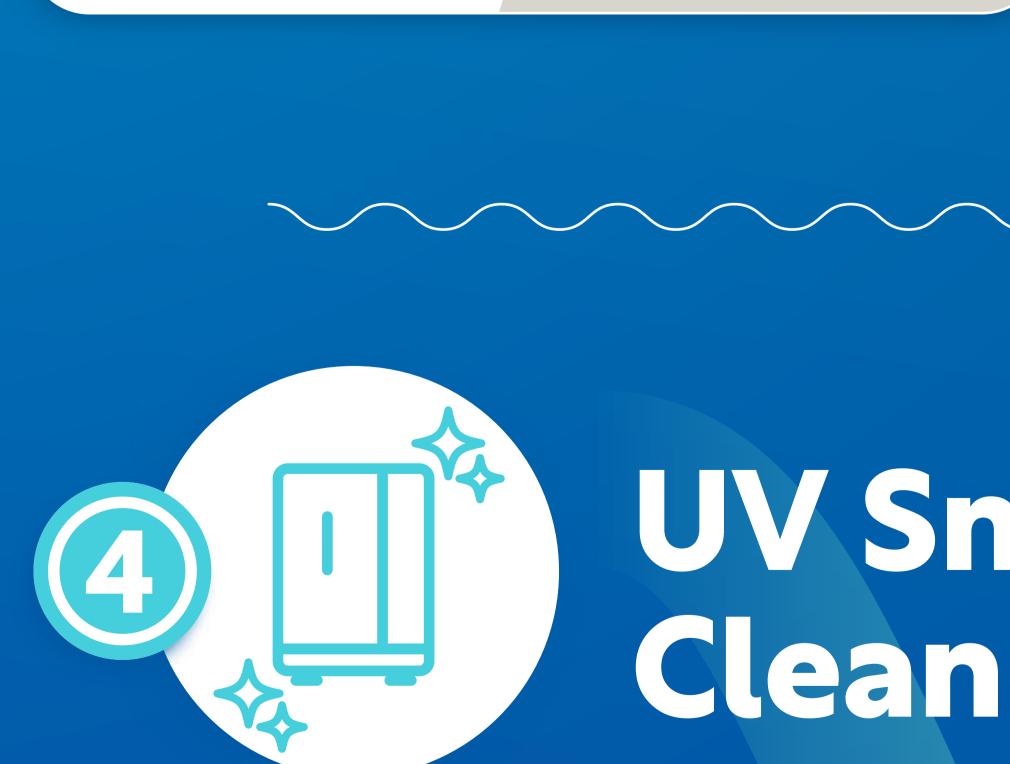



Complete control questions

Gently insert the probe into

of the ultrasound probe.

Scan staff badge and RFID tag

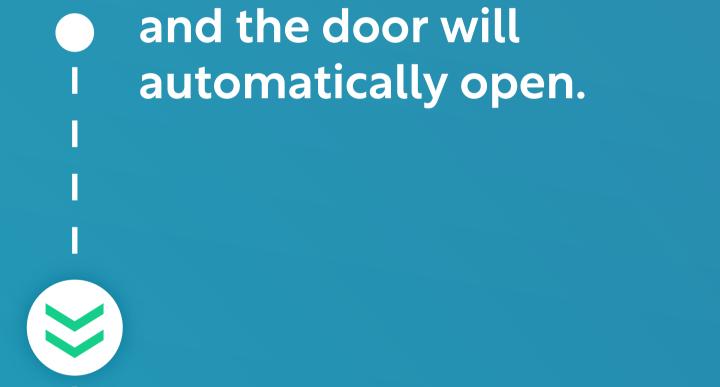

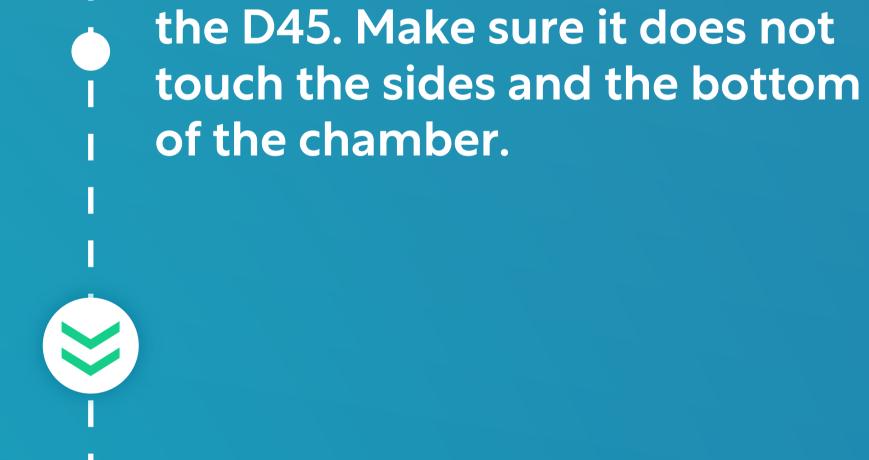


Scan your staff badge to open the D45.

Start the disinfection process

which only takes few seconds.


The D45 notifies you when the

disinfection is completed.

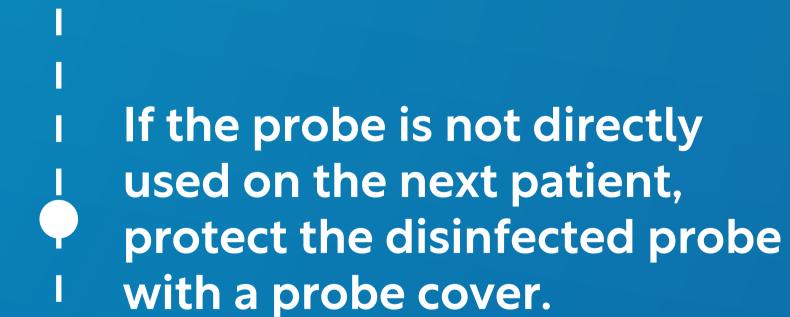


Carefully take out

the ultrasound probe.

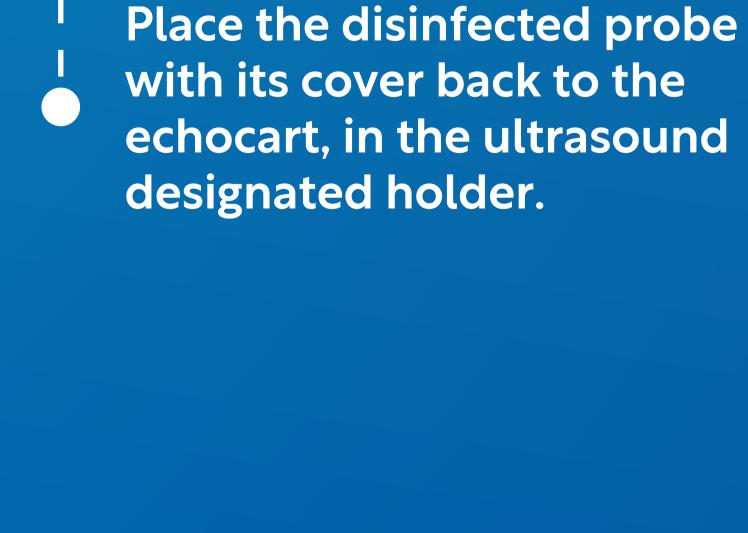



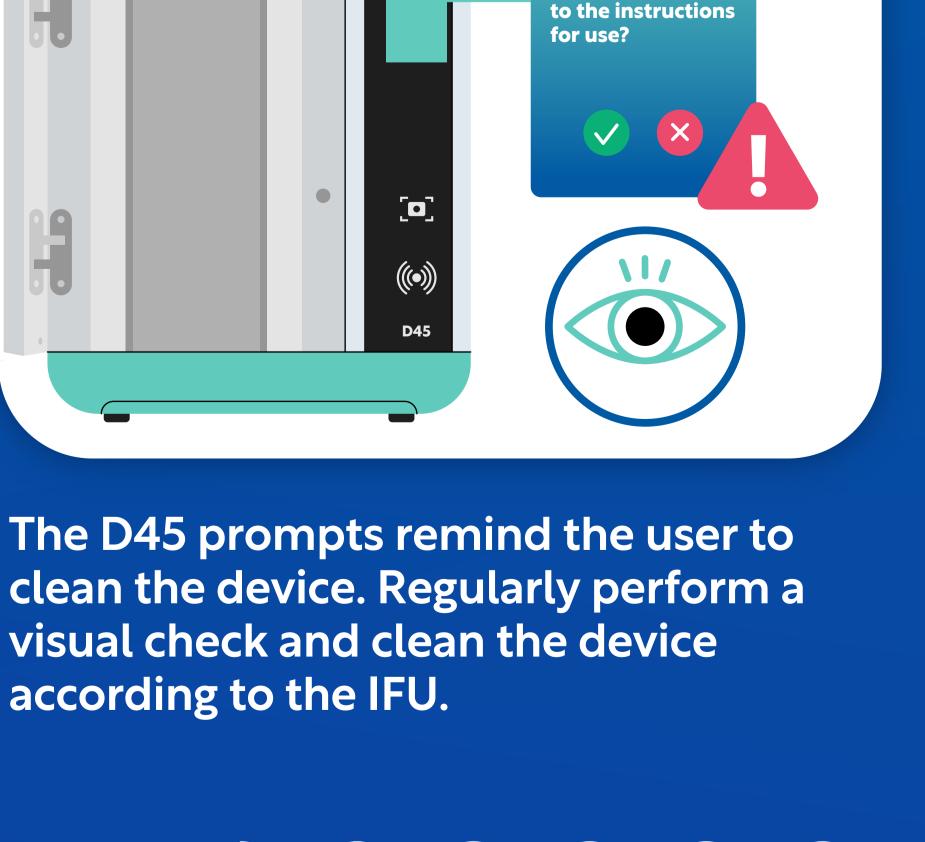



Clean the internal and external surfaces

of the D45 with a microfiber wipe and

UV Smart Device Cleaner.



validation stickers.

Validation Sticker





for validation traceability.

D45 needs cleaning Did you clean

the D45 according