

Canada's most pure, stable, and effective hypochlorous eye care solution.




BIHOCL O.D. is an effective-yet-gentle solution that can be used regularly in your eyelid and eyelash routine. It's an effective complement to existing treatment protocols for dry eye disease, blepharitis, MGD, and similar conditions, and can be used multiple times daily to set the foundation or optimal daily eye hygiene (eyegiene). Plus, BIHOCL O.D. is a scientifically formulated, 100% pure hypochlorous acid solution with antimicrobial, antibacterial, and antiviral properties.


Just like oral hygiene, a strong preventative approach is critical to maintaining ocular health — that's where BIHOCL O.D. comes in, for excellent eyegiene.




OPTOMETRIST & OCULAR HYGIENIST
RECOMMENDED

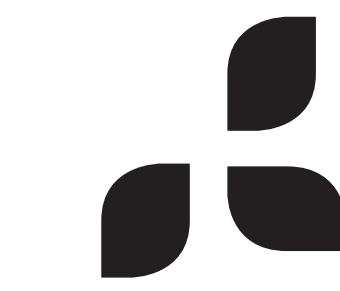


ALL NATURAL

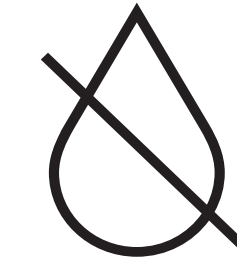


OIL FREE

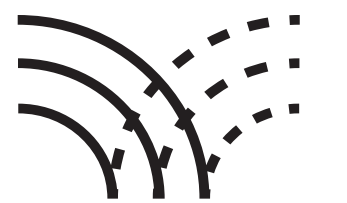


VEGAN & ORGANIC

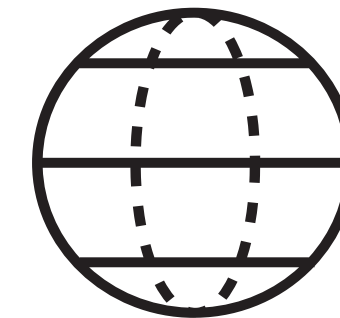


ECO-SAFE

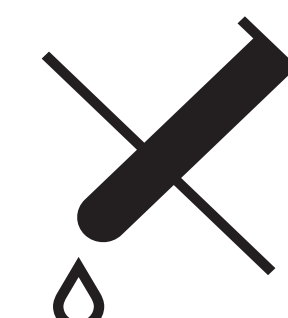


ADDITIVE FREE

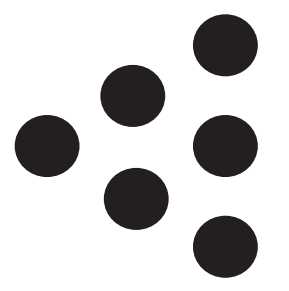

EASY APPLICATION

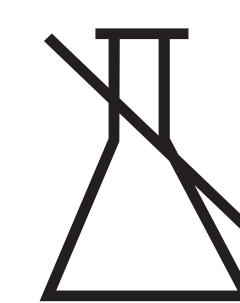

ALCOHOL FREE

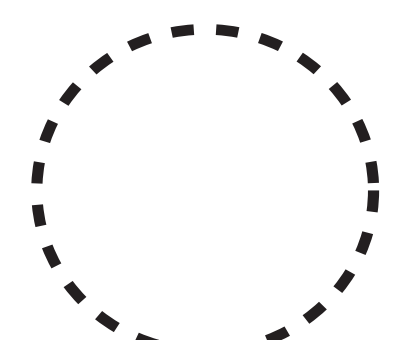

HIGHLY EFFECTIVE

Using BIHOCL O.D.

Remove contact lenses and/or cosmetics, and wash your face and hands before proceeding.



Spray twice on closed eyelid and brow area




Gently rub eyelid and eyelash roots in a lateral or circular motion using cotton round or clean hands until debris is removed (optional)



Do not rinse and let air dry, with eyes open



Repeat as needed for the second eye

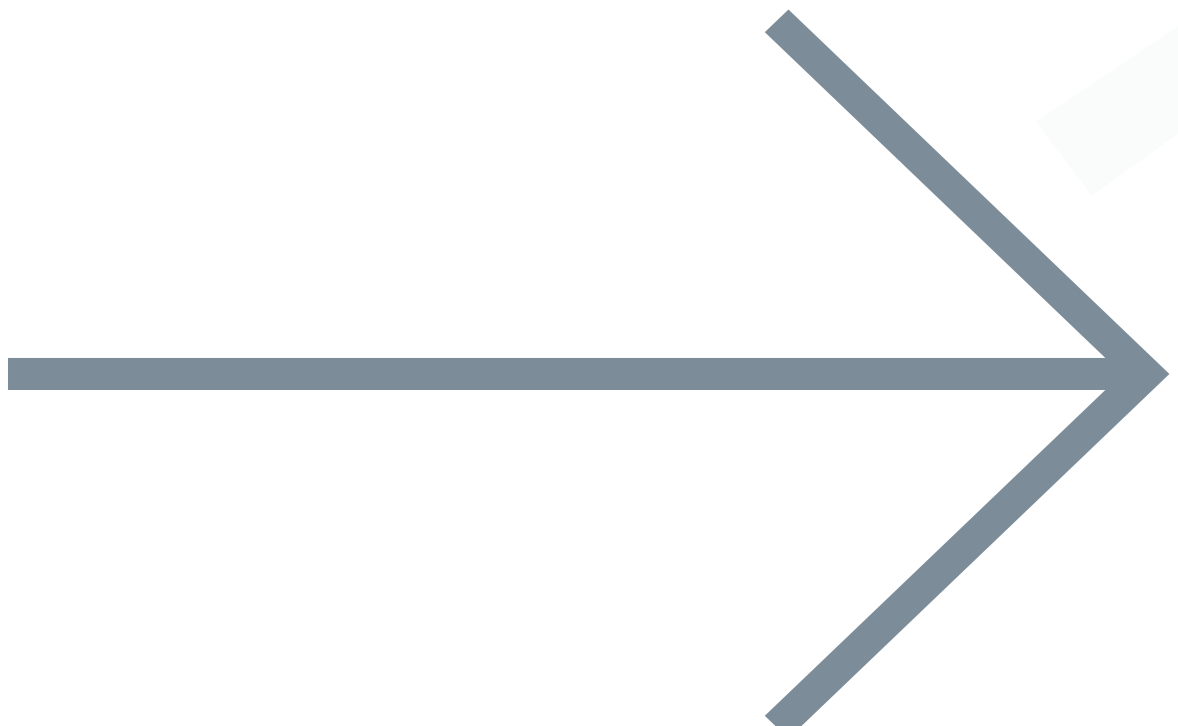

BIHOCL O.D. can also be safely misted onto the ocular and periocular region. A combination of both techniques can be utilized to fully cleanse the eyelid and eyelash. BIHOCL O.D. is gentle enough for everyday use on sensitive mucosal membranes, making it ideal as a critical part of eye care protocols.



What is hypochlorous?

Hypochlorous (HOCl) is the active ingredient in BIHOCL O.D. It's a powerful component of our bodies' internal defense system, produced by our white blood cells. Using state-of-the-art technology, BIHOCL O.D. is an electrolyzed saline and hypochlorous formulation that is safe and effective as a daily regimen within any dry eye protocol.

BIHOCL O.D. is the only hypochlorous formulation tested by the National Institute of Health (NIH) for purity and is unmatched in its combination of safety, efficacy, and stability.

- Strictly controlled pH (4-5)
- 2 year shelf life, 6 months once opened
- Available in 2 fl. oz. (60 mL) and 4 fl. oz. (118 mL)

Learn more at
www.biomiq.health