PURECLEANSE™

STAT

Instructions for Use

PureCleanse™ STAT

QuickTab™ Cleansing Lavage & Debridement System
for Wounds, Skin and Burns

FOR HOSPITAL & MEDICAL PROFESSIONAL USE

Rev 001

REF STAT250

Product Description

PureCleanse™ STAT is a safe, effective irrigation, moisturizing, and debridement solution formulated to actively cleanse wounds and prepare optimal wound bed healing conditions. The product's unique packaging is quick and easy to open, easy to squeeze, and allows for rapid application of the pure hypochlorous acid (HOCl) cleansing solution contained within.

Indications

Under the supervision of health care professionals, PureCleanse™ STAT is indicated for the cleansing, irrigating, moistening, debridement, and removal of foreign material including microorganisms and debris from exudating and/or dirty wounds, acute and chronic dermal lesions such as stage I-IV pressure ulcers, partial and full-thickness wounds, diabetic foot ulcers, post-surgical wounds, first/second degree burns, grafted and donor sites, and minor irritations of the skin in addition to moistening and lubricating absorbent wound dressings. The product is safe for use on all ages.

Contraindications

Avoid use on individuals allergic or sensitive to hypochlorous acid (HOCl).

Storage

Store the product in a room temperature environment and away from direct sunlight, heat, or extreme cold. Do not freeze. For adequate efficacy, do not use the solution past its expiration date.

Disposal

Dispose of remaining PureCleanse™ STAT solution. Place plastic bottle in the regular garbage or biohazard waste disposal as per your facility's standard protocol. The bottle is made of high density polyethylene (HDPE), a recyclable material. Please recycle wherever possible.

Ingredients

Water, sodium chloride, and 0.033% hypochlorous acid (330 PPM) as a preservative to inhibit growth of microorganisms within the solution.

Instructions

To prepare for use, twist the QuickTab™ cap until it snaps off the bottle's pointed nozzle. Invert and squeeze the bottle to dispense the contained solution.

The following are instructions for common use cases of the product.

Cleansing a Wound


Apply PureCleanse™ STAT solution liberally to irrigate wound and peri-wound tissue. Ensure the wound is thoroughly saturated. No rinse is required.

Soaking a Wound

Saturate gauze with PureCleanse™ STAT solution and apply to wound, soaking for minimum 5 minutes, up to 15 minutes. Remove the gauze. For further debridement, use the gauze to wipe the wound bed. No rinse is required.

Packing a Wound

Saturate packing gauze with PureCleanse™ STAT solution and place it into the wound as appropriate. No rinse is required.

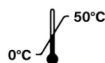MD

Rx ONLY

MADE IN CANADA

Questions, comments or concerns? Please contact us.

info@biomiq.health | 1-800-573-1295 | www.biomiq.health

Manufactured for Biomiq Inc. by Sterasure Inc.

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Fabriqué pour Biomiq Inc. par Sterasure Inc.