FOR HOSPITAL/MEDICAL PROFESSIONAL USE,
AND FOR OVER-THE-COUNTER USE

Rev 002

REF G3003

Product Description

BIHOCL™ PureGel™ is an ultra-pure HOCl formulation in the form of a hydrogel wound cleanser. This isotonic hydrogel promotes the occlusion of wounds and enhances the lubrication of wound dressings. Gentle and effective, BIHOCL™ PureGel™ aids in the mechanical removal of debris and foreign materials from the wound or application site. As the clear hydrogel gently moves across the wound bed, it facilitates the cleansing process, creating a favourable environment for wound healing. Trust the advanced properties of BIHOCL™ PureGel™ for optimal wound care and management.

Intended Use

BIHOCL™ PureGel is a hydrogel designed for cleansing, occlusion, moistening, debriding, and effectively removing foreign materials, including microorganisms and debris, from exuding and/or dirty wounds. This advanced solution caters to acute and chronic dermal lesions, ranging from stage I-IV pressure ulcers, stasis ulcers, diabetic ulcers, and post-surgical wounds to first and second-degree burns, abrasions, and minor skin irritations. BIHOCL™ PureGel also finds application in conditions such as diabetic foot ulcers, ingrown toenails, grafted and donor sites, and exit sites, providing comprehensive care. Its exceptional properties make it ideal for occluding wounds and lubricating absorbent wound dressings, ensuring optimal wound management. Trust BIHOCL™ PureGel™ to deliver superior performance in wound care under professional guidance.

Ingredients

Hypochlorous Acid (0.03%), Electrolyzed Saline, Mineral Clay, Sodium Phosphate

Safety

Not for injection. For external use only. Discontinue use immediately if irritation, sensitization, or reaction occurs. Do not use on patients with known sensitivity to hypochlorous acid (HOCl). This product is non-pyrogenic and is not made with natural rubber latex.

Disposal

BIHOCL™ PureGel™ is an environmentally conscious solution, leaving behind no harmful residues or by-products. After use, it naturally breaks down into harmless elements of salt and water, ensuring its safe disposal through the public sewer system.

Instructions

If necessary, debride the wound or cleanse using a wound cleanser like BIHOCL™ PureCleanse™. Apply a generous amount (approximately 1/4" to 1/2" thick) of BIHOCL™ PureGel™ to cover the entire wound bed, including areas of undermining. Apply a thin layer to the skin around the wound and allow it to dry. Proceed to cover with an appropriate wound dressing or covering, avoiding silver and other wound dressings containing heavy metals.

For general skin care, cleanse the affected area with a suitable skin cleanser such as BIHOCL™ PureCleanse™. Once dry, apply a thin layer of BIHOCL™ PureGel™ and reapply as needed.

For indwelling vascular catheters, apply a sufficient quantity of BIHOCL™ PureGel™ to fully cover the skin area around the catheter. Then, cover with an appropriate site dressing.

For ostomy care, apply a thin layer of BIHOCL™ PureGel™ to the peristomal area and allow it to dry before applying the ostomy appliance.

Over-The-Counter (OTC) Use

BIHOCL™ PureGel™ is also available for OTC use, intended to facilitate mechanical cleansing and removal of dirt, debris, and foreign material from skin abrasions, lacerations, minor irritations, cuts, exit sites, and intact skin.

Storage

Do not freeze. Store at room temperature up to 25°C, or refrigerate. Dispense from original packaging only. For optimal efficacy, do not use the solution past its expiration date (printed on the bottle itself).

Health Canada Class II Medical Device License (MDL) #110242

Questions, comments or concerns? Please contact us.

info@biomiq.health | 1-800-573-1295 | www.biomiq.health

BIOMIQ

Manufactured for Biomiq Inc. by Sterasur Inc.

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Fabriqué pour Biomiq Inc. par Sterasure Inc.